



Disposable Automatic Intraosseous Injector

Intravascular Access in seconds



Adult depth is adjustable by 4 different anatomical locations.



Pediatric depth is adjustable by age in years.



1 Position the B.I.G. 90° to the bone



2 Remove Safety Latch



3 Trigger



4 Clip & Secure



5 Remove Stylet



6 Aspirate*



7 Flush*



8 Infuse Drugs / Fluids



* Bone Marrow may not always be present.

*10-20cc in Adults / 5-10cc in Pediatrics.